



OasisSpace

**Upright Rollator Walker
(OS-9223L)**

Instruction Manual



**** Please read the instruction carefully before using.**

Thank you for choosing OasisSpace Stand Up Rollator Walker. Wish you happy with it !

YOU ARE ONE OF OUR VALUED CUMSTOMERS!

We hope to receive your review or any suggestions for us to improve our products.

Need More Help?

Email: oasisspace.csr@gmail.com

NOTICE FOR USE

1. Do not install or use this equipment without first reading and understanding this instruction sheet.
If you are unable to understand the cautions or instructions, please contact a healthcare professional or technical personnel before attempting to install this equipment.
2. Before using the Rollator Walker, you should be trained by a healthcare professional or dealer until you have practiced with someone and you feel confident you can maneuver by yourself.
3. Please consult with your physician or therapist to determine proper adjustment and usage.

OasisSpace

PRODUCT INFORMATION

OasisSpace upright walker is delivered in the box. After unpacking Please keep the box for future storage.

If the package or product is damaged, please contact us.

** Product Overview



1	2 x Hand Brake
2	Stable Balance Bar
3	Seat Belt
4	Foldable Widen Seat
5	Rollator Walker Bag
6	2 x 10"Front Wheel
7	2 x Rear Wheel
8	2 x Armrest
9	1 x Cane Holder

**** Product images only for reference purpose, so please check the subject produce with "Package Content"**

** Package Content

1. OS-KLD9223L	x1 set (Silver or Red)
2. Instruction Manual	x1
3. Accessories	x1 Rollator bag x 1 Screw set

QUICK USING GUIDE

**** INSTALL**

Step 1. Unfold the Seat and lock it to the frame

Turn the walker to the upright position. Press down the sides of the seat until you hear a “click”, indicating the seat is locked to the frame.

Double check it is firmly installed.



Step 2. Install the wheel frames to the main frame

Please note that the front and rear wheels are integrated in the wheel frame.

The left side of wheel frame is marked as “L” while the right side of wheel frame is marked as “R”.

Attach the “L” and “R” wheel frame to the main frame, which has marked with “L” and “R” too.

Insert the plastic knobs and metal washers to their original positions. Make sure the plastic knobs are well tightened.

The wheel frames should be locked.

Double check it is firmly installed.




Step 3. Install the adjustable Handles

- Install the adjustable handle bars
 - Adjust the height according to your desired height
- Handlebars should be set at wrist level to keep your back straight.



Step 4. Install the Hand Brake

- Insert the handbrake to the handle bar twist on the screw.
- Remember to hold the brake tightly when you are walking.
- The handgrip can be adjusted: it can be rotated 360 degrees, the front and rear distance can be adjusted. user could adjust it to a comfortable position according to the actual needs.
- Double check it is firmly installed.

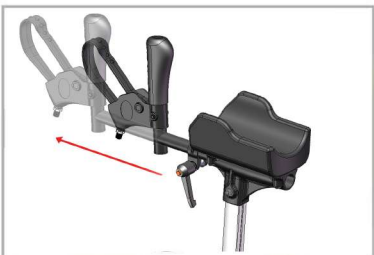

Step 5. Install the Shopping Bag

- Place the rollator bag on the front tiller on each side.
- It can keep necessary essentials at hand easily.



Step 6. Check Everything in Box while you Go Out



** Storage

To fold and storage the Rollator Walker, you just operate the install step oppositely.

SAFETY PRECAUTIONS

- * Please check the brake, wheels before each use.
- * The rollator should be used on a level, dry ground. Ensure that you are accompanied by another person if possible. Practice using the rollator until you feel comfortable.
- * When you use the rollator please keep your hands touching the brake. It can ensure you to stop the rollator immediately and effectively reduce the risk of falling.
- * Avoid sudden braking otherwise the rollator will slip especially on the wet smooth surface.
- * If possible please find somebody to help you to go up stairs, take the elevator and e.g.
- * The maximum weight capacity is 450 lbs and 11 lbs for mesh bag.
- * It is not recommended to use the rollator in BAD WEATHER (Rain/Storm/Lighting e.g.) or on the smooth/wet ground.
- * Please slow down when start heading downhill.
- * Do not use the rollator on stairs or an escalator.
- * Do not attempt to push the rollator while someone is sitting in the seat.
- * Do not use the seat to transport people or objects.
- * Do not use the seat without the backrest in place.
- * Do not sit on the seat when the rollator is on an incline.
- * Only use the rollator as a walking support.
- * Only use the specified accessories and spare parts for this rollator
- * Loads should only be transported using the basket. Do not hang heavy items from the handles as this may affect the stability of the Rollator.
- * Do not self-propel the rollator while seated. Serious injury to the user or damage to the rollator frame or wheels may result from improper use.
- * Do not use the rollator to walk backwards.
- * Caution must be taken when negotiating curbs and other obstacles, or when using on sloping, uneven or slippery surfaces. Failure to do so may result in a serious risk of a fall or injury.
- * Do not perform any adjustments to the rollator while it is in use.
- * All wheels must be in contact with the floor at all times.
- * Brakes must be in the locked position before using the seat.
- * When using the rollator in a stationary position, the brakes must be locked.

CLEAN AND STORAGE

- * The frame part can be wiped with a damp cloth or washed with neutral cleaning agent.
- * The wheels can be cleaned with a damp brush with plastic bristles (DO NOT use wire brushes)
- * The seat back, seat and transporting pouch can be washed with a mild soap.
- * Do not use a high pressure or steam cleaner.
- * Dry the rollator with a towel. If possible use some machine oil on the parts of brake to avoid rusting such as brake cable, screw, e.g.

PRODUCT SPECIFICATION

Size and Weight	
Seat size	22.8×11 in
Seat height	26.1 in
Total width	30.6 in
Total length	30.2 in
Total height/Height of handle (adjustable at 6 levels)	42.3-46.9 in
Empty weight (include belt and mesh bag)	23.37 lbs
Rollator weight Capacity	450 lbs
Shopping bag weight Capacity	11 lbs
Folding size	30×26.2×39.8 in
Wheels	
Front wheels	10×13.7 in
Back wheels	8×14.2 in
Materials	
Frame	Aluminum (corrosion protection)
Seat	Nylon, black
Seat back	Nylon, black
Tire	PU+Aluminum
Operating Conditions	
Temperature	0 – 122°F
Humidity	20 - 80 %
Storage Conditions	
Temperature	-4 bis + 140 °F
Humidity	45 - 60 %

LIMITED WARRANTY

This warranty is extended only to the original purchaser /user of our product. The warranty does not cover commercial use (such as rental or institutional use) and is not transferable.

OasisSpace warrants that it's free of defects in material and workmanship and that, when operated within the guidelines and restrictions of this manual, will remain so free of defects and in material and workmanship for a period of One(1)Year from the original date of purchase.

This warranty doesn't extend to those items which may require replacement due to those items which may require replacement due to negligence, abuse, misuse, contamination, accident, improper operation, maintenance, or storage, or any circumstances beyond the control of OasisSpace.

The warranty shall not apply to problems arising from normal wear or failure to follow instructions or maintain the product. Failure to follow the instructions and warnings can result in personal injury or product damage and will void product warranty.

This warranty does not extend to products modified without OasisSpace's expressed written consent and any unauthorized modification will void this warranty.

This warranty does not include any labor charges incurred for the repair of the product or replacement part installation. Do not return defective products without prior written authorization. Transportation costs and shipping damage while submitting for product repair or replacement are responsibilities of the purchaser, not OasisSpace. However, the return shipping cost to the customer will be prepaid.

The foregoing warranty is exclusive and in lieu of all other warranties expressed or implied including, but not limited to, the implied warranty of merchantability and fitness for a particular purpose. OasisSpace will not be liable for any consequential or incidental damages whatsoever.

OasisSpace