

User Manual

1. GENERAL EXPLANATION OF YOUR ROLLATOR



1	Push handle	6	Backrest
2	Brake lever	7	Brake cable
3	Base frame	8	Shopping bag
4	Rear wheel	9	Frame for front wheels
5	Front fork	10	Front wheel

Your rollator is equipped with various components and parts. Learn this before you proceed with studying this manual. Designs and specifications may change without prior notice.

2.ASSEMBLE YOUR ROLLATOR

When you first receive the rollator, assemble before the rollator in use. Below is a summary of the individual parts of the ROLLATOR, see picture 1.



Picture 1

The following describes how you should use the rollator.

2.1 Assembly of the front wheels

- ✓ Place the frame with the front wheels in the receipt tubes, see picture 2; Move the frame with the front wheels in the desired height and press the silver button in, see picture 3, on the rollator;
- ✓ You will hear a click which confirms that it is well assembled; Repeat the above steps also on the opposite side of the rollator.



Picture 2



Picture 2

2.2 Assembly of the rear wheels

- ✓ Place the rear wheel in the reception tube, see picture 4;
- ✓ Press the silver button in, see picture 5;
- ✓ Move the rear wheel to the desired height and let the silver button out;
- ✓ You will hear a click this confirms that the back wheel properly attached;
- ✓ Repeat the above steps also on the opposite side of the rollator.



Picture 4



Picture 5

Warning:

Make sure the brake cables outside of the frame are not interfering the rear wheels, to prevent crush injuries.

2. 3 Height Adjustment

- ✓ Place the rollator on the front and rear wheels;
- ✓ Place the handle tube in the reception tube push handle, see picture 6;
- ✓ Move the push handle to the desired height and let the silver button out;
- ✓ Repeat the above steps also on the opposite side of the rollator. Keep the same height on both sides.



Picture 6

2.4 Assembly of the backrest

- ✓ Place the backrest in the receipt part, see picture 7;
- ✓ Place the backrest into the receipt part on the rollator;
- ✓ Turn the clip on the plastic part from open to lock, , see picture 8 and 9, which confirms that it is well assembled;
Repeat the above steps also on the opposite side of the backrest.



Picture 7



Picture 8



Picture 9

2.5 Walk with your rollator

- ✓ When you walk with your rollator make sure you walk upright, with the rollator near you body, see Figure 1. This is for the best support and the safest movements. It will give you the correct body posture.

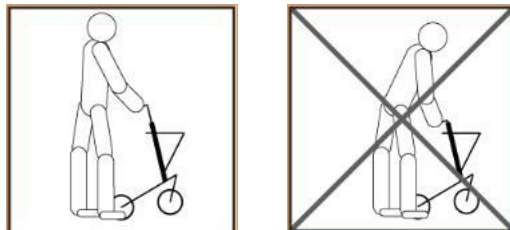


Figure 1

2.6 On folding and unfolding of the rollator

Folding the rollator

- ✓ Your rollator can be folded up. This allows you to easily carry your rollator and to transport.
- ✓ Make sure that you are behind the rollator;
- ✓ Grab the bar below the seat, see picture 10 and move it up;
Bring the front and back of the rollator together, see picture 11;
Folding the backrest up.



Picture 10



Picture 11

Unfold the rollator

- ✓ Make sure that you are behind the rollator;
- ✓ Folding the backrest;
- ✓ Push the seat down, see picture 12;
- ✓ Make sure that the frame is fully expanded, see picture 13.



Picture 12



Picture 13

2.6 Brakes

- ✓ Your rollator has two types of brakes. You can slow down while driving by using the brake, you can also stop the rollator if you stopped on the parking brake

Slowing down

- ✓ When you want to reduce speed while you are behind your rollator is running, switch the brake in squeeze, see picture 13;
- ✓ Please note that you do not overpush the rollator while holding the brake squeezes. which will be risk the damage of tires and the brakes;
- ✓ Do not use the rollator if the brakes are defective!



Picture 13



Picture 14

Parking brake use

- ✓ Push the parking brake down ,as soon as you hear a click that is from the parking brake of the rollator, it is parked. see picture 14;
- ✓ Pull up the parking brake after rest to walk, see picture 13. You will hear a click again once the parking brake is pulled up.

2.7 Sitting

- ✓ Make sure the rollator is on the parking brake set before you sit on the seat of your rollator;
- ✓ When you sit down please make sure that your feet are between the rear wheels positioned.see picture 15



Picture 15

For disassembly of your rollator, please follow the steps above in reverse order.